



Instructions

1. Attach GelShot adaptor to the head the ultrasound applicator with hinge at 12 o'clock position.
2. Gently lift the adaptor retaining ring from side opposite of the hinge, and mist with water.
3. Remove a tab from the tray by tearing along perforations, peel back liner, and remove GelShot by applying pressure to back side of the disc reservoir to gently lift it out.
4. Insert GelShot disc, flat side down, into the recessed area of the adaptor. Press down to remove any air bubbles between GelShot and transducer.
5. Align GelShot to center of the adaptor.
6. With GelShot in place and no air bubbles present, lower the retaining ring and gently press down to latch it in place.
7. Verify that the GelShot is secured and in place.
8. Mist both the GelShot and treatment site with water. Repeat as necessary.
9. Perform therapy.

10. At the conclusion of the treatment, lift up on the retaining ring and remove the GelShot.
11. Dispose of the GelShot using normal waste disposal methods.
12. Clean the adaptor and transducer with anti-microbial wipe, and dry.

Warnings

- **DO NOT** use GelShot from an open or damaged package.
- **DO NOT** use GelShot if the disc is not clear or contains particulate.
- **DO NOT** use if the GelShot is dried out.
- **DO NOT** use GelShot over irritated or damaged skin.
- **DO NOT** use GelShot for more than one treatment.
- **DO NOT** store GelShot pads below 40°F or above 135°F.
- **DO NOT** autoclave GelShot adaptor.
- GelShot Therapeutic Ultrasound Coupling Discs are Non-Sterile.