

Instructions

1. Remove AutoGel pad from the packaging by peeling back liner, running your finger along the edge of the pad, then prying up on the side of the pad.
2. Make sure both sides of the AutoGel pad are wet (if not, simply spray/rinse both sides with tap water).
3. **For Vertical Placement:**
Place the AutoGel pad into the AutoSound gel well and apply to the body area with fixation straps.
For Horizontal Placement:
Place AutoGel Pad onto treatment area. Cover AutoGel with AutoSound, fitting AutoGel into AutoSound well. Secure with fixation straps.
4. Perform therapy.
5. When treatment is finished discard the AutoGel Pad using normal waste disposal method (No special handling required).

Warnings



DO NOT use AutoGel pad from an open or damaged package.

- **DO NOT** use AutoGel pad if it appears damaged or contains particulate.
- **DO NOT** use if the AutoGel pad is dried out.
- **DO NOT** use the AutoGel pad over irritated or damaged skin.



DO NOT reuse. Single patient. Single use.



DO NOT store AutoGel pads below 40°F or above 135°F.



AutoGel pads are non-sterile.



Richmar[®]
Engineering Tomorrow's
Healthcare

Toll Free 1.888.549.4945
International +1.423.648.7730
4120 South Creek Road
Chattanooga, TN 37406 USA
www.Richmarweb.com

Richmar is an ISO 13485 Certified Company.
3048.16.02.A ©2016 Richmar **Made in USA**